Inhalt

Content Contenido

Vorkörner
Initial bur
Fresa
inicial
(2 Stck. / 2 p. /
2 uds.)

Spiralbohrer
Twist drills
Broca helicoidal
(je 2 Stck. / 2 p. of each size/
respectivamente 2 uds.)

Pin
Pin
Chincheta
BMP00: 34 Stck. /
34 p. / 34 uds.
BMPBA: 10 Stck. /
10 p. / 10 uds.

Notice: Only 10 Pins in
Master-Pin-Control Basic


CE 0044

Fig.	186RF	203RF	203RF	MP10 (*)
Shank ¹	204	205	205	
Size ²	018	006	008	
Length mm	12.0			3,5

¹ 204=RA, 205=RA L ² Largest working part diameter in 1/10 mm

⊕ Minimaldurchmesser Minimal diameter Diámetro mínimo
⊖ Außendurchmesser External diameter Diámetro exterior

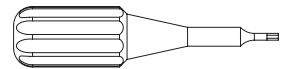Schraubendreher zum Entfernen der Pins
Screw driver to remove pins
Atornillador para retirar las chinchetas
illustrated 1:1

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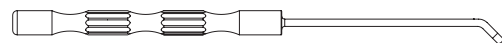
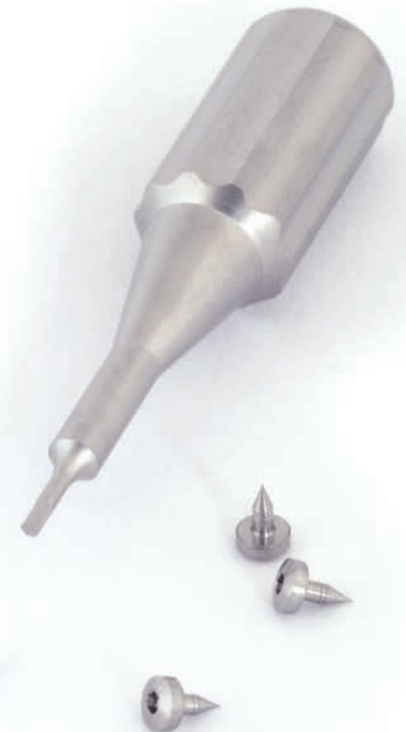
Master-Pin-Control Master-Pin-Control Basic

Revolutionary Hybrid Pin System

85FLBM08 - 0315

Bone Management® is a registered trademark of the Hager & Meisinger GmbH, Germany

Hager & Meisinger GmbH | Hansemannstr. 10 | 41468 Neuss | Germany
Tel.: +49 (0) 21 31 20 12-0 | Fax: +49 (0) 21 31 20 12-222 | www.meisinger.de | info@meisinger.de
Meisinger USA, L.L.C. | 10200 E. Easter Avenue | Centennial, Colorado 80112 | USA
Tel.: +1 (303) 268-5400 | Toll free: +1 (866) 634-7464 | Fax: +1 (303) 268-5407
www.meisingerusa.com | info@meisingerusa.com

Master-Pin-Control Master-Pin-Control Basic

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The Master-pin-Control Bone Management® system is used for the fixation of membranes (absorbable, non-absorbable and titanium membranes) in order to avoid micro-mobility of the graft. Thanks to the sharp tips and the stable construction, the pins can be precisely inserted into cortical bones. The pins have an additional mini-thread and are a hybrid composed of a screw and pin. The thread makes easy and safe removal possible after a healing period has taken place. This thread enlarges the surface of the shaft at the same time and therefore clearly offers more stability. Master-Pin-Control Basic is the entry-level system for the safe fixation of membranes while Master-Pin-Control Professional makes a larger number of pins available.

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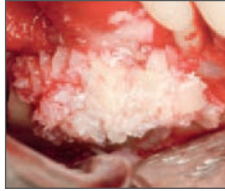
The twist drills included within the scope of supply make the setting of bleeding points possible when using bone replacement material and can also be used as a pilot bur in the case of very cortical bone.

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After applying the augmentation material, the selected membrane is attached to the present defect, adjusted to the graft, and fixed with the pins. Important: A compression of the membrane must be avoided.

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Using the MP12 pin holder, the hybrid pins are applied into the cortical bone. The pins provide for secure fixation. The MP14 fixation holder can be used in order to make loosening the pin from the pin holder easily.

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After a healing period has taken place, the pins can safely

and easily be loosened with the aid of screw driver. NOTE: The pins are a disposable product and are therefore not to be used again.

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Pin Box: The sterilisation of the Pins can be carried out in the Pin Box tray directly. The Pin Box or the Pins can also be taken from the set and individually sterilized. To open the Pin Box, turn the swivel lock on the side of the set.

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- Dentoalveolar growth which has not come to an end (exception: cases in which no dentoalveolar growth can be expected, e.g. ectodermal dysplasia)
- Active infections as well as pathological processes
- Insufficient bone quality and diseases that impair bone metabolism

A NOTE ON MEMBRANE TECHNOLOGY: In this connection, please pay attention to the information provided by the respective manufacturer of the membrane selected by you.
NOTE: Related contraindications and further instructions deemed set-specific and generally valid can be found at the download area of our website at "Application and Safety Instructions for Bone Management® Systems".

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